EYENIMAL

A LIVING ADVENTURE

ELECTRONIC FLEA COMB

PEIGNE ELECTRONIQUE ANTI-PUCES

- FR - Guide d'utilisation
- EN - User's guide
- ES - Manual de instrucciones
- D - Gebrauchsanleitung
- IT - Manuale d'uso
- NL - Gebruikershandleiding

Declaration of conformity
EYENIMAL Electronic Flea Comb NGDIVEYE001 – NGDIV001 – NGDIVEYE001

FCC Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any changes or modifications to this equipment not expressly approved by NUM'AXES may cause, harmful interference and void the FCC authorization to operate this equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Responsible party: EYENIMAL INC
 World Financial District
 60 Broad Street – Suite 3502
 New York, NY 10004
 sales@eyenimal.com

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User's guide

COMB CHARACTERISTICS

This comb delivers an electrical current just strong enough to kill fleas, yet without bothering the animal. The negative ions and active oxygen are released at both ends of the comb. They quickly penetrate into the animal's fur to eliminate foul smells.

The negative ions are also able to dissipate the electrostatic phenomenon on your pet's fur, which in turn leaves a shinier coat.

Precautions prior to use:

- Designed to be used solely on a perfectly dry coat, the comb is very sensitive to humidity.
- It is recommended to use the comb before giving your pet a bath.
- Never use this flea comb device on another animal without disinfecting it first.
- Stop using the comb if its teeth are deformed, as this might cause the device to short-circuit.
- Avoid all direct contact with comb teeth whenever the comb is turned on.
- Return the protective cover to its place following use or cleaning in order to protect comb teeth.

Energy

- Runs with 2 "AA LR06" batteries (not included).

Maintenance

Disinfect the device and comb teeth after each use with a cloth soaked in 70% alcohol (be sure to turn the device off during cleaning and wait to make sure it is completely dry before the next use).

COMB USE INSTRUCTIONS

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

1. Hold the handle and open the battery compartment, insert the two "AA" batteries (see Figure 1), then put the comb back together.

Fig. 1

2. Press the ON / OFF button on the side of the handle; you'll hear a buzz indicating that the comb is working correctly and ready for use. You have the option of using the device like a regular comb by leaving the "electronic comb" function turned off.
3. Keep a 45° angle between the comb and your pet's fur (see Figure 2 below) and avoid touching your pet's eyes, ears or chops. The black rubber-coated fixtures on the teeth ends must be positioned on the lower side of the fur. Move the comb slowly through the coat, preferably one zone at a time, for a more efficient result. The brushing direction may be bottom to top or top to bottom, but be sure to maintain the 45° angle at all times.

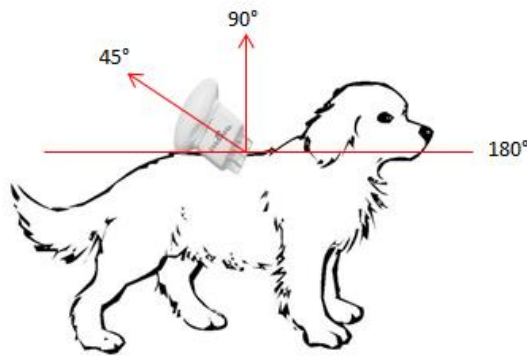
Fig. 2

Fleas typically begin to infest your cat or dog on the backside before working their way down the back onto the abdomen, at which point they spread over the animal's entire body.

4. When a flea is detected, the buzz stops temporarily. It will only resume once the comb has been cleaned with a small cleaning brush included with the device (Figure 3).

Fig. 3

To maximize the efficiency of this device and extend its battery life, limit each period of operation to 5-10 minutes.

Reminder: To avoid receiving an electrical shock, do not hold the comb on both sides (see Figure 4).

Fig. 4

WARRANTY

A. Troubleshooting

Should your product stop working or develop a fault, first read this user's guide over, then check the batteries and replace them if necessary and retry. Also check that you are using the product correctly.

If the problem persists, contact your distributor or check out on www.numaxes.com. You can also contact NUM'AXES at **+33.2.38.69.96.27** or by Email **export@numaxes.com**.

Depending on the extent of the malfunction, you may have to return your product for service and repair.

For all repairs, please supply the following:

- the complete product
- the proof of purchase (invoice or sales receipt)

If you omit one of these items, the After Sales Service will have to invoice you for any repair costs incurred.

B. Warranty

NUM'AXES guarantees the products against all manufacturing defects for 1 year after purchase. All postage and packing charges will be the purchaser's sole responsibility.

C. Warranty conditions

1. The warranty will be effective only with a proof of purchase (invoice or sales receipt). The warranty is exclusive to the original purchaser.
2. This warranty does not cover the following:
 - changing the battery,
 - direct or indirect problems incurred when sending the product back to the distributor or to NUM'AXES,
 - damage to the product caused by:
 - negligence or incorrect use (ex: bites, breakage, cracking),
 - misuse from not following directions
 - repairs performed by unauthorized persons.
 - loss or theft.
3. If the product is found to be defective, NUM'AXES will either decide to repair or to replace it.
4. No claim may be made against NUM'AXES, particularly in relation to incorrect use or breakdown.
5. NUM'AXES reserves the right to modify the characteristics of its products with a view to making technical improvements or to complying with new regulations.
6. The information contained in this guide may be subject to amendment without prior notice.
7. Photos and drawings are not contractual.

D. Disposing or recycling of your device at the end of its life

The pictogram  which appears on your product means that this one can not be thrown with household refuse. You have to bring the device to a collection location suitable for treatment, development, recycling of electronic wastes or bring it back to your distributor.

Adopting this process, you do something for the environment; you contribute to the preservation of natural resources and to the protection of human health.

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